### DERMORISOLV REPAIR

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Ingredients (INCI): Polyperfluoromethylisopropyl Ether, Panthenol, Polyperfluororethoxymethoxy Difluoroethyl PEG Phosphate, Tocopherol, Tocopheryl Acetate, Olus Oil, Glycerin, Ethylhexyl Palmitate, Glyceryl Stearate, C12-20 Acid PEG-8 Ester, Cetearyl Alcohol, Dimethicone, Lecithin, Ascorbyl Palmitate, Citric Acid, Sodium Phytate, Benzyl Alcohol, Dehydroacetic Acid, Sodium Hydroxide, Aqua.

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FABBRICANTE: KALIS SRL Via Caodevilla, 38 - Pederobba (TV) - MADE IN ITALY

### DERMORISOLV REPAIR

Numerous environmental factors (exposure to the sun, cold or irritating agents such as surfactants, soaps, metal powders, etc.) or pathologies (eczema, contact or frictional dermatitis, allergic dermatitis) can alter the skin's normal state of wellbeing and reduce its physiological barrier function, leading to an increase in epidermal permeability. When this happens, the skin becomes vulnerable, easily exposed and reactive to external environmental agents, resulting in the onset of symptoms such as dryness, irritation and inflammation.

Dermorisolv Repair is a special vegetable oil-based emulsion containing a high concentration of the film-forming agent PFPE, panthenol and vitamin E. Thanks to its special formulation, the medical device Dermorisolv Repair creates a non-clogging barrier effect, which limits the loss of moisture in the skin and protects it from contact with irritating or sensitising external agents, promoting the restoration of physiological skin function and alleviating bothersome symptoms such as dryness (xerosis), irritation and redness.

### INDICATIONS

Dermorisolv Repair is a rebalancing barrier cream indicated as a specific treatment for the control and attenuation of skin symptoms due to inflammation, irritation and dryness of the skin, in particular in the following cases:

- treatment and prevention of dermatitis of various origins, such as contact dermatitis and/or dermatitis from physical agents (e.g. allergic, atopic, frictional, diaper dermatitis);
- treatment and prevention of dermatitis caused by radiotherapy, dermatological treatments, UV lamps, chemical peels;
- barrier protection in all cases of dehydration, dryness or cracking of the skin on the face, hands and body;
- rebalance of the skin condition altered by various factors (e.g. skin dryness due to pharmacological treatments).

### HOW TO USE

Apply Dermorisolv Repair on clean skin, directly on the affected parts, lightly massaging to distribute the cream. Apply once or several times a day as needed, for as long as necessary until the symptoms disappear or as advised by your doctor.

As a barrier cream, apply before contact with irritating agents.

If symptoms do not improve after two weeks of treatment, consult your doctor.

### WARNINGS

- The product is for external use. Apply on intact skin.
- In case of concomitant pharmacological therapies, consult your doctor in advance.
- Do not use in case of hypersensitivity or individual allergy to one or more components of the product.
- Prolonged use of products for topical use may lead to sensitisation; if this occurs, it is advisable to stop treatment and consult a doctor.
- Keep the product out of the reach and sight of children.
- Do not use the device beyond its expiry date.
- Do not use in case of damaged packaging.
- It is recommended to use the product within 6 months from the first opening.

### STORAGE

Store in a cool, dry place, away from heat sources and light.

### SIDE EFFECTS

There are no known or expected side effects related to the use of the product. Tests carried out on the product to date have shown no sensitisation or other side effects.

### COMPOSITION

Ingredients (INCI): Polyperfluoromethylisopropyl Ether, Panthenol, Polyperfluoroethoxymethoxy Difluoroethyl PEG Phosphate, Tocopherol, Tocopheryl Acetate, Olus Oil, Glycerin, Ethylhexyl Palmitate, Glyceryl Stearate, C12-20 Acid PEG-8 Ester, Cetearyl Alcohol, Dimethicone, Lecithin, Ascorbyl Palmitate, Citric Acid, Sodium Phytate, Benzyl Alcohol, Dehydroacetic Acid, Sodium Hydroxide, Aqua.

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